



GIMA

PROFESSIONAL MEDICAL PRODUCTS

Gima S.p.A.

Via Marconi, 1 - 20060 Gessate (MI) Italy

gima@gimaitaly.com - export@gimaitaly.com

www.gimaitaly.com

TYPE 4 MULTI-VARIABLE STEAM STRIPS - DUAL

ATTENTION: *The operators must carefully read and completely understand the present manual before using the product.*

PRODUCT NAME

IS 2401 Multi-variable Indicator Strips - Class 4

PRODUCT DESCRIPTION

PMS Steri-test Class 4 indicator strips, having a dimension of 13 mm x170 mm with heat and steam sensitive ink, are printed on white carton, film coated, each one having two parts. Indicator signs have rectangular shape with rounded corners, 3 items arranged in a line. Indicator ink turns into reference color after an effective complete cycle. Strips are in Class 4 category according to ISO 11140-1 standard.

PURPOSE

PMS Steri-test Class 4 indicator strips operate by means of color change and indicate that the loads have been exposed to sterilization process or not. They also have a function of indicating the conformity of sterilization conditions, (considering possible incorrect packaging, improper loading of the device and device failures.)

APPLICATION








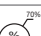
1. Place min. one indicator strip for each sterilization pack. Placing the strip to a location where the steam reaches difficultly, helps you get better results
2. Place the sample into the autoclave, operate the device at 134°C 3,5 minutes, in accordance with operation instruction.
3. Remove the indicator and observe the color changes after a complete operation cycle.
4. Operation cycle is successful, if the indicator color reaches the reference color or darker.
5. Operation cycle is insufficient, if the indicator color is lighter than reference color.

WARNINGS

Use of biological and mechanical indicators, besides chemical indicators, are recommended to validate the effectiveness of sterilization.

STORAGE CONDITIONS

Store at max. 35°C, dry place, and 70% RH max. The product has a shelf life of 3 years when stored at recommended conditions. No special storage conditions are necessary after sterilization.

Simboli - Symbols				
	Data di fabbricazione Date of manufacture	REF	Codice prodotto Product code	 Conservare al riparo dalla luce solare Keep away from sunlight
	Leggere e seguire attentamente le istruzioni per l'uso Read instructions carefully	LOT	Numero di lotto Lot number	 Conservare in luogo fresco ed asciutto Keep in a cool, dry place
	Data di scadenza (vedi scatola / bustina) Expiration date (see box / package)		Dispositivo monouso, non riutilizzare <i>Disposable device, do not re-use</i>	 Limite superiore di temperatura Upper limit of temperature
	Limite umidità Moisture limitation			

REF 35954 / IS2401

Fabbricato da/Manufactured by:
PMS TIP TEKNOLOJILERI SANAYİ VE TİCARET A.Ş.
Mersin Tarsus Organize Sanayi Bölgesi 12. Cad.No:2
33020 Mersin/TURKEY
Made in Turkey

