

See It. Share It. Save It.



FOR VETERINARIANS

Welch Allyn Digital MacroView™ Veterinary Otoscope

The Welch Allyn Digital MacroView Veterinary Otoscope is a simple, affordable tool that lets you see, explain, and save digital images of the ear.

Offering you clear, live and still images of the ear canal and tympanic membrane, the Digital MacroView Veterinary Otoscope is a valuable tool for the treatment and management of your patients' ears. Easy to learn and use, the Digital MacroView is a plug and play system requiring no additional purchase of software.

Here's how the Welch Allyn Digital MacroView Veterinary Otoscope can benefit you:

- > Improves communication which can lead to better client compliance
- > Gives you an excellent training aid for educational purposes
- > Offers you the ability to view small details of ears, mouth and teeth
- > Allows you to store digital images in patient records for enhanced documentation of exam findings
- > Provides you the ability to share images with other providers as visual descriptive reference




WelchAllyn®

Advancing Frontline Care™

The Welch Allyn Digital MacroView™ Veterinary Otoscope

Features you expect from an otoscope and more:

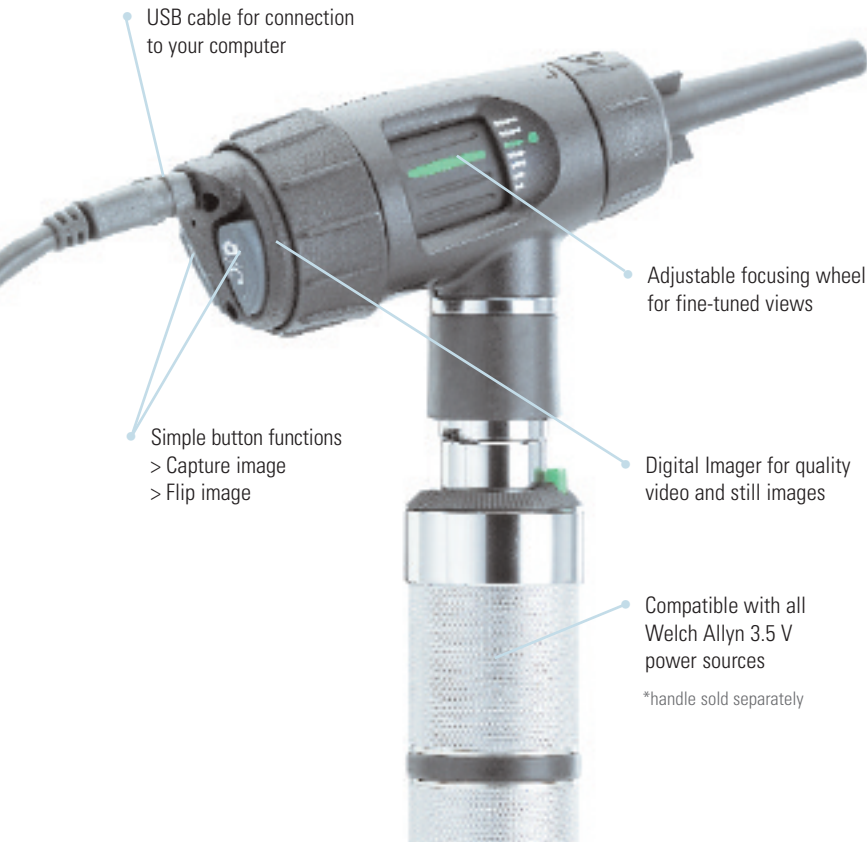

USB cable for connection to your computer

Adjustable focusing wheel for fine-tuned views

Simple button functions
> Capture image
> Flip image

Digital Imager for quality video and still images

Compatible with all Welch Allyn 3.5 V power sources

*handle sold separately

Improve compliance through client involvement

	Before Treatment	After Treatment
EARS Show abnormalities such as mites, foreign bodies, and inflammation prior to treatment.		
MOUTH Show airway disorders, lesions, and injuries in the mouth.		
DENTAL Show dental disorders that are normally difficult to view when explaining treatment options.		

USING THE DIGITAL MACROVIEW VETERINARY OTOSCOPE IS EASY AS 1 – 2 – 3

1 Plug In.

Plug the otoscope into your computer using the USB cable.

2 Power On.

Put on ear specula and turn on light.

3 Play.

View live picture or capture image.

Ordering Information

23960 Digital MacroView Veterinary Otoscope with 10 ft USB cable and set of five reusable ear specula (including instrumentation tip)

Accessories

23492	Additional set of five Reusable Specula (4 mm, 4 short, 5 mm, 5 short and 7 mm instrumentation tip)	06500-U	Lamp
23834-S	4 Short Veterinary Reusable Speculum	23900-10	Replacement USB Cable, 10 ft
23835-S	5 Short Veterinary Reusable Speculum	71000-A	NiCad Rechargeable Handle (Plug-in)
23834	4.0 mm Veterinary Reusable Speculum	71000-C	NiCad/Convertible Rechargeable Handle (Plug-in)
23835	5.0 mm Veterinary Reusable Speculum	71670	NiCad Rechargeable Handle for Universal Charger
23837	7.0 mm Veterinary Reusable Speculum	71900	Lithium Ion Rechargeable Handle with AC Charging Module
23847	7.0 mm Veterinary Instrumentation Speculum	71910	Lithium Ion Rechargeable Handle for Universal Charger

System Requirements: Windows® XP Service Package 2 or higher with an available USB 2.0 port.

To find out how the new Welch Allyn Digital MacroView Veterinary Otoscope can help you see, explain and save digital images of the ear, go to www.welchallyn.com/dvm or call 1.800.535.6663.

4341 State Street Road, Skaneateles Falls, NY 13153 USA
(p) 800.535.6663 (f) 315.685.3361 www.welchallyn.com/dvm

WelchAllyn®

Advancing Frontline Care™