

## HOW TO USE THE SHIRT CORRECTLY?

This easy to put on and off MPS-Medical Pet Shirt® is specially designed for the anatomy of cats and can therefore only be used for cats.

### Important information

Consult your veterinarian when you want to use the Medical Pet Shirt® for medical purposes. Your veterinarian can recommend a suitable treatment. Keep monitoring your cat regularly to make sure the fit is not too tight. Keep your cat under supervision of a responsible adult at all times while using the Medical Pet Shirt®. Please consult your veterinarian if your cat's behavior is changing or when your cat has excessive licking/chewing habits.

### Washing instructions

For longer usage, it is possible that the print of the Medical Pet Shirt® and the label could fade over time. With the washing instructions below, this process can be delayed. Washing, maintaining and handling of delivered items always remains the responsibility of the customer.

#### Instructions:

- Always turn inside out for washing.
- Fasten all press studs before washing.
- We recommend using a laundry bag.
- Wash the Medical Pet Shirt® up to a maximum of 90°C/194°F.
- Never put the Medical Pet Shirt® in the dryer! Hang it out and let it air dry.
- Do not iron directly over the print and never too hot.
- Do not use bleach or other chemicals to clean the Medical Pet Shirt®.

Using bleach and other chemicals can cause irritations to your cat's wound, surgical area, hotspots or more...!

**TIP!** Little (peeing) accidents can happen while wearing the Medical Pet Shirt®. To ensure that your cat remains protected, two Medical Pet Shirt® are often purchased. While one Medical Pet Shirt® can be washed, the other shirt can be worn. When your cat needs to wear the Medical Pet Shirt® longer than a week, it's always better to have a back-up Medical Pet Shirt®!

### Specifications

The Medical Pet Shirt® for cats is toxic/lead free and is made of:

