

HOW TO USE THE MPS-MEDICAL PET SHIRT® CORRECTLY?

This easy to put on and take off MPS-Medical Pet Shirt® is specially designed on the anatomy of dogs. Therefore, it can only be used for dogs.

IMPORTANT INFORMATION:

Consult your veterinarian when you want to use this product for medical purposes. Your veterinarian can recommend a suitable treatment. Keep monitoring your pet regularly to make sure the fit is not too tight. Keep your pet under supervision of a responsible adult at all times while using this product. However, please consult your veterinarian if your pet's behavior is changing or when your pet has excessive licking/chewing habits.

WASHING INSTRUCTIONS:

For longer usage, it is possible that the print of the MPS-Medical Pet Shirt® and the label could fade over time. With the washing instructions below, this process can be delayed. Washing, maintaining and handling of delivered items always remains the responsibility of the customer.

Instructions:

- Always turn inside out for washing.
- Fasten all press studs before washing.
- We recommend using a laundry bag.
- Wash the MPS-Medical Pet Shirt® up to a maximum of 90°C/194°F.
- Never put the MPS-Medical Pet Shirt® in the dryer! Hang it out and let it air dry.
- Do not iron directly over the print and never too hot.
- Do not use bleach or other chemicals to clean the MPS-Medical Pet Shirt®.

Using bleach and other chemicals can cause irritations to your dog's wound, surgical area, hotspots or more...!

TIP! Little (peeing) accidents can happen while wearing the MPS-Medical Pet Shirt®. To ensure that your dog remains protected, two MPS-Medical Pet Shirt® are often purchased. While one MPS-Medical Pet Shirt® can be washed, the other MPS-Medical Pet Shirt® can be worn. When your dog needs to wear the MPS-Medical Pet Shirt® longer than a week, it's always better to have a back-up shirt!

SPECIFICATIONS:

The MPS-Medical Pet Shirt® is toxic/lead free and is made of:

