

## HOW TO PUT THE SHIRT ON YOUR CAT?

We know how important it is to be able to quickly dress and undress your cat, which is why our MPS-Medical Pet Shirt® is made of soft and stretchy high-quality material. The MPS-Medical Pet Shirt® has a snug fit to prevent your cat from wiggling out. It is important however to keep monitoring your cat if the snug fit isn't too tight. Getting the MPS-Medical Pet Shirt® on and off your cat is simple, so let's take a look!

**Step 1:** Roll up the shirt with the middle head opening in your hand. Make sure that the light blue inner layer is facing down. Insert your cat's head through the opening.

**Step 2:** Followed by inserting one leg into the provided smaller leg opening and the other leg into the other remaining smaller opening.

**Step 3:** Now pull the MPS-Medical Pet Shirt® over your cat's body all the way to the back.

**Step 4:** The lower belly flaps must be pulled between the hind legs towards the upper part near the base of the tail.

**Step 5:** Secure the two flaps with the snaps/poppers. We advise to close all snaps/poppers to keep the shirt perfectly in place.

**Step 6:** Take a good look around the legs, neck and tail to see if there is enough room to move and breathe!

