

HOW TO PUT THE SHIRT ON?

We know how important it is to be able to quickly dress and undress your rabbit, which is why our MPS-Medical Pet Shirt® is made of soft and stretchy high-quality material. The MPS-Medical Pet Shirt® has a snug fit to prevent your rabbit from wiggling out. It is important however to keep monitoring your rabbit if the snug fit isn't too tight. Getting the MPS-Medical Pet Shirt® on and off your rabbit is simple, so let's take a look!

- **Step 1:** Roll up the shirt with the middle head opening in your hand. Make sure that the blue inner layer is facing down. Insert your rabbit's head through the opening, followed by inserting one leg into the provided smaller leg opening and the other leg into the other remaining smaller opening.
- **Step 2:** Now pull the shirt over your rabbit's body all the way to the back. The lower belly flaps must be pulled between the hind legs towards the upper flaps near the base of the tail.
- **Step 3:** Secure the two flaps with the buttons/snaps to each other. Take a good look around the legs, neck and tail to see if there is enough room to move and breathe! You can adjust the snaps if needed.