

HOW TO PUT THE SHIRT ON?

We know how important it is to be able to quickly dress and undress your dog. Therefore, our MPS-Medical Pet Shirt® is made of soft and stretchy high-quality material. The MPS-Medical Pet Shirt® has a snug fit to prevent your dog from wiggling out. However, it is important to keep monitoring your dog if the snug fit isn't too tight. Getting the shirt on and off your dog is simple, so let's take a look!

Step 1: Roll up the shirt with the middle head opening in your hand. Make sure that the light blue inner layer is facing down. Insert your dog's head through the opening.

Step 2: Followed by inserting one leg into the provided smaller leg opening and the other leg into the other remaining smaller opening.

Step 3: Now pull the MPS-Medical Pet Shirt® over your dog's body all the way to the back.

Step 4: The lower belly flaps must be pulled between the hind legs towards the upper part near the base of the tail.

Step 5: Secure the two flaps with the snaps/poppers. We advise to close all snaps/poppers to keep the shirt perfectly in place.

Step 6: Take a good look around the legs, neck and tail to see if there is enough room to move and breathe!

