[www.draminski.com](http://www.draminski.com)

*iScan*

Quick Reference Guide

## I. INTRODUCTION

We are giving you the *iScan* ultrasound scanner, which is designed for field veterinarians and specialists dealing with reproduction of large animals.

*iScan* provides an excellent and clear image, which makes the diagnosis much easier. Durable and robust housing makes it possible for the *iScan* to be used in the harshest conditions. The ultrasound scanner is very light and handy, which makes working with it very easy and convenient.

Read instruction manual carefully before starting the ultrasound scanner. It will guarantee safe usage and long, reliable functioning of the device.

**NOTE: Please read the complete Instruction Manual located on the supplied CD.**

## II. FUNCTION KEYS DESCRIPTION

1. Image freeze. Rescan.
2. Turning the device off (after pressing and holding the button).

G1 + signal amplification adjustment in the closer field

G2 + signal amplification adjustment in the further field

On/off switch

1. Opening of menu and accepting chosen option, confirming settings
2. Confirming marker position when measuring

Exit menu

Navigation keys:

= scrolling through menu

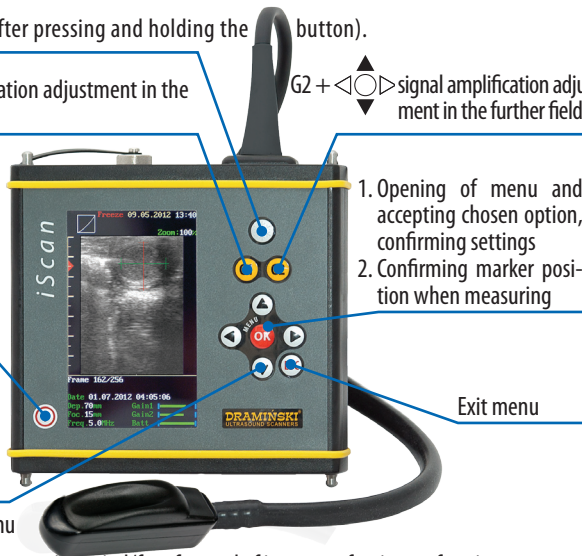
= „Cineloop” function automatic rewind/fast-forward of image – after image freezing

= „Cineloop” function manual advance frame by frame – after image freezing

= setting markers when measuring, enter name and number of the animal during saving the images and Cine

= setting focus marker at the optimal depth

= setting scanning range (from 4 to 12 cm)

### III. MOST COMMONLY USED FUNCTIONS during an examination

IMAGE FREEZING		AFTER PRESSING THIS BUTTON THE IMAGE IS FROZEN TURNING THE DEVICE OFF (AFTER HOLDING THE BUTTON)
MEASURING	 	<b>MENU / MEASURING</b> AFTER FREEZING THE IMAGE CAN BE MEASURED BY USING CURSORS (MARKERS). YOU CONFIRM THE MARKER POSITION WITH THE OK BUTTON.
AMPLIFICATION ADJUSTMENT	 	SIGNAL AMPLIFICATION ADJUSTMENT. GAIN 1 BUTTON  = AMPLIFICATION IN THE CLOSER FIELD. GAIN 2 BUTTON  = AMPLIFICATION IN THE FURTHER FIELD.
CHANGE OF PENETRATION DEPTH		PENETRATION RANGE FROM 4-12 CM
FOCUS		SETTINGS OF MANUAL FOCUS ARE ADJUSTED WITH NAVIGATIONAL BUTTONS.
FILTERING SIGNAL FROM CONVERTERS		<b>MENU FREQUENCY</b> ADJUSTMENT IN THE RANGE OF BETWEEN 4MHZ TO 9 MHZ
CINELOOP		FUNCTION ALLOWING TO REPLAY MAXIMUM OF 256 LAST FRAMES SINCE THE MOMENT OF IMAGE FREEZING.  YOU CAN MANUALLY MOVE SEQUENCES FRAME BY FRAME  YOU CAN AUTOMATICALLY REPLAY CINE LOOP FORWARDS/BACKWARDS
ZOOM		<b>MENU/ZOOM/</b> 120%, 140%, 160%
B+B MODE		<b>MENU/MODE</b> DIVISION OF THE SCREEN INTO TWO SECTIONS
SAVING IMAGES AND CINE		<b>MENU/IMAGE OR CINE</b> FUNCTION ENABLES QUICK SAVING OF IMAGE OR CINE LOOP 256 FRAMES TO MEMORY
GAMMA		<b>MENU/GAMMA</b> FUNCTION ENABLES ADJUSTMENT OF BRIGHTNESS AND CONTRAST

## IV. DEVICE AND PROBE MAINTENANCE

The *iScan* ultrasound scanner should be cleaned with a wet cloth soaked with a delicate detergent solution directly after use. While cleaning, the socket should be closed with the cap attached to the housing. Before wet cleaning the battery pack should be disconnected from the device.

After wet cleaning the scanner should be wiped dry and packed into the transport case.

Probe can be cleaned with running (not hot) water or immersed in water with an addition of detergent or properly dissolved disinfectant.

**Do not use highly concentrated agents and scrubbing agents. During cleaning, always protect contacts and sockets from moisture.**

## V. OPERATION NOTES

### **Signs of unusual behaviour of device**

Checking actions

#### **Lack of power – the device does not turn on**

- Check if the pack is properly connected or the cleanliness of contacts.
- You can also check proper operation with a different pack.

#### **Image too bright or too dark**

- Check amplification and GAMMA level adjustments

#### **Lack of charging indicator in charger**

- Check cable connections
- Check power network

#### **Short work time on battery**

- Battery not charged
- Low ambient temperature
- Battery is exhausted (normal sign, resulting from the battery's construction)

If none of the basic actions helps, please contact our service at the DRAMIŃSKI Company,

**tel. +48 89 527 11 30 or e-mail: [info@draminski.com](mailto:info@draminski.com)**

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