



Care instructions

We have built the Easi-Scan remote display to be rugged and splash resistant, however, it does contain sensitive electronics. With a little care and maintenance, you will get many years of excellent imaging.

The Easi-Scan Remote Display will be damaged if you:

- Drop it
- Hose it down
- Wipe the screen with your hand

For a long and happy life:

- Clean immediately after use — before any debris dries.
- Grit/mud should first be removed by rinsing gently — do not rub the screen.
- Dry excess water off the housing with clean soft cloth — do not rub the screen.
- Only wipe the screen with a clean cloth.
- Secure in protective carry case or neoprene sleeve once clean and dry.

Specifications

Display size:	8.2" or 21 cm diagonal
Luminance:	800 cd/m ²
Contrast ratio:	1000:1
Battery life:	5 hours
Power source:	Easi-Scan battery
Environmental proofing:	IP65
Temperature range:	-20 °C to +50 °C or -4 °F to +122 °F
Weight (with Easi-Scan battery):	1.65 kg / 3.64 lb
To be used with:	Easi-Scan and Duo-Scan

The Easi-Scan Remote Display has been designed to operate as a component of BCF ultrasound scanners (now known as IMV imaging ultrasound scanners) and as such the combined system conforms with the requirements of European Directive 89/336/EEC relating to electromagnetic compatibility (EMC) and also 2002/95/EC relating to Reduction of Hazardous Substances (RoHS). If you require any further information regarding these directives, please contact us directly.

Note: All IMV imaging products are to be used for animal applications only.

Call **+44 (0)1506 460 023** to
arrange your demonstration.

Visit www.imv-imaging.co.uk for
more information.

Follow us     